HUNTLEIGH

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Mode d'emploi

Bruksanvisning

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aanwijzing

INSTRUCTIONS FOR USE

Bruksar

alimatları

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**Hydroven 12/
Hydroven 12 Lymphassist
Arm, Leg and Insert Garments**

To be used only under the direction of a physician.
100% Latex Free.


For use with **Huntleigh Hydroven® 12 / Hydroven® 12 LymphAssist** pump only.

Refer to the **Hydroven 12 / Hydroven® 12 LymphAssist** User Manual for complete information on using the pump and garments.


Garments and Garment Inserts


Arm Garment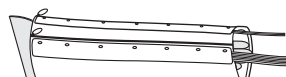

Arm Garment with Insert

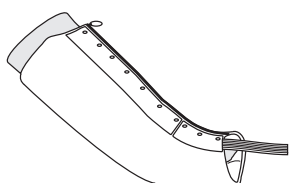Leg Garment

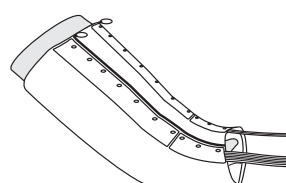Leg Garment with Insert

Applying the Garments

1. Make sure the garment is the correct size for the limb, and that a garment insert is used if necessary.
2. Undo the zip on the garment.
3. If a garment insert is fitted to the garment, fully fasten one of the zips between the garment and insert, leaving the other unfastened.
4. Before applying the garment (and insert, if fitted) to the limb, zip up the first 150 mm (6") of the unfastened garment zip. Put the garment (and insert) onto the limb and fully fasten the zip.
5. Make sure the patient is in a comfortable position with the limb supported or elevated as necessary.
6. Connect the garment(s) to the two push-fit, polarised 12-way connectors on the pump. If only one garment is used, fit the rubber cover to the lower pump connector.
7. Connect the pump to a suitable mains power supply. Switch on the pump.
8. Refer to the Hydroven 12 User Manual for further information on changing the therapy settings on the pump control panel.

Warnings

- Do not apply the garment to the limb unless it is partially zipped, as you may damage the garment zip.
- Do not undo the zips or attempt to remove the garments during the therapy session or you may damage the zips. Make sure the therapy session has stopped and the garments have deflated before you remove the garments.
- Do not stand or walk with the garments fitted.
- In the event of a power failure or fault whereby the garment remains inflated, disconnect the tubeset(s) in order to deflate the garment(s) and then remove the garment(s) from the limb(s).

Cleaning the Garment

Wipe down using a neutral detergent or soap powder at 51°C (120°F).

Do not iron


Do not dry clean

Do not tumble dry

Do not machine wash. Air dry thoroughly.

Disinfecting the Garment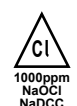

After cleaning, wipe complete garment over using 70% isopropyl alcohol wipe or a chlorine-releasing agent at 1000 p.p.m. available chlorine.

Do not use phenol or phenol-derivative disinfectant.

Rinse with clean water to remove any residue.
Air dry thoroughly.

Cleaning the Tubeset

Use a soft brush. Air dry only.

Do not immerse in water. Do not machine wash.

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If a serious incident occurs in relation to this medical device, affecting the user, or the patient then the user or patient should report the serious incident to the medical device manufacturer or the distributor.
In the European Union, the user should also report the serious incident to the Competent Authority in the member state where they are located.

Manufactured for Huntleigh Healthcare Ltd on behalf of;

ArjoHuntleigh AB
Hans Michelsensgatan 10
211 20 Malmö, Sweden

www.huntleigh-diagnostics.com/

Huntleigh Healthcare Ltd.

35 Portmanmoor Road, Cardiff, CF24 5HN, United Kingdom
T: +44 (0)29 20485885 sales@huntleigh-diagnostics.co.uk
www.huntleigh-diagnostics.com

www.huntleigh-healthcare.us/

Registered No: 942245 England & Wales. Registered Office:
ArjoHuntleigh House, Houghton Hall Business Park, Houghton Regis, Bedfordshire, LU5 5XF
©Huntleigh Healthcare Limited 2014

A Member of the Arjo Family

® and ™ are trademarks of Huntleigh Technology Limited

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice.

HUNTLEIGH