HUNTLEIGH

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Mode d'emploi

Bruksanvisning

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aanwijzing

INSTRUCTIONS FOR USE

Bruksan

alimatları

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Hydroven 1 / Hydroven 3 Arm, Leg and Insert Garments

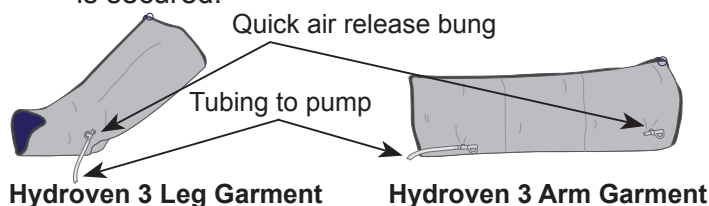
To be used only under the direction of a physician.
100% Latex Free.

For use with **Huntleigh Hydroven® 3** pump only.
Refer to the **Hydroven 3** User Manual for complete information on using the pump and garments.


Before fitting the garment ensure all quick air release bungs are closed, as this will effect the efficiency of the garment.

Applying the Garment

1. Check the type of garment - Hydroven 1 (single chamber) or Hydroven 3 (three chamber) - and the length of the garment required.
2. Undo the zip on the garment.
3. If a garment insert is fitted to the garment, fully fasten one of the zips between the garment and insert, leaving the other unfastened.
4. Before applying the garment (and insert, if fitted) to the limb, zip up the first 150 mm (6") of the unfastened garment zip. Put the garment (and insert) onto the limb and fully fasten the zip. Make sure that the quick air release bung is secured.

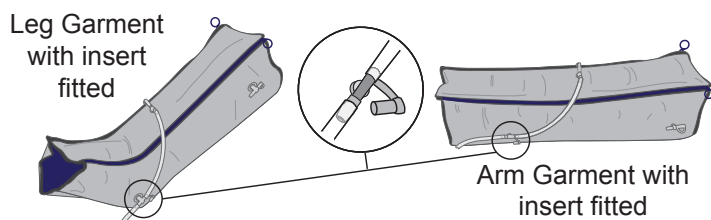5. Make sure the patient is in a comfortable position with the limb supported or elevated as necessary.
6. Check that the insert piece connecting tube is not kinked and is attached to the garment using the lower outlet bung as shown below.

7. Attach the garment tubing to the pump ensuring a "click" is heard from each snap-lock connector.
8. If only one garment is to be used, attach the garment to either port on the pump. The system will automatically identify that only one garment is to be used.
9. Switch on the pump and adjust the pressure control accordingly (refer to the pump user manual).

Warnings

- Do not apply the garment to the limb unless it is partially zipped, as you may damage the garment zip.
- Do not undo the zips or attempt to remove the garments during the therapy session or you may damage the zips. Make sure the therapy session has stopped and the garments have deflated before you remove the garments.
- Do not stand or walk with the garments fitted.
- In the event of a power failure or fault whereby the garment remains inflated, disconnect the tubeset(s) in order to deflate the garment(s) and then remove the garment(s) from the limb(s).

Cleaning the Garment

Wipe down using a neutral detergent or soap powder at 40°C (104°F).

Do not iron

Do not dry clean

Do not tumble dry

Do not machine wash. Air dry thoroughly.

Disinfecting the Garment

After cleaning, wipe complete garment over using 70% isopropyl alcohol wipe or a chlorine-releasing agent at 1000 p.p.m. available chlorine.

Do not use phenol or phenol-derivative disinfectant.

Rinse with clean water to remove any residue.
Air dry thoroughly.

Cleaning the Tubeset

Use a soft brush. Air dry only.

Do not immerse in water. Do not machine wash.

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NORGE

ArjoHuntleigh Norway AS
Olaf Helsets vei 5
N-0694 OSLO
Tel: +47 22 08 00 50
Faks: +47 22 08 00 51
E-mail: no.kundeservice@arjohuntleigh.com

DANMARK

ArjoHuntleigh A/S
Vassingerødvej 52
DK-3540 LYNGE
Tel: +45 49 13 84 86
Fax: +45 49 13 84 87
E-mail: dk_kundeservice@arjohuntleigh.com

USA

ArjoHuntleigh Inc.
2349 W Lake Street Suite 250
US-Addison, IL 60101
Tel: +1 630 307 2756
Free: +1 800 323 1245 Institutional
Free: +1 800 868 0441 Home Care
Fax: +1 630 307 6195
E-mail: us.info@arjohuntleigh.com

CANADA

ArjoHuntleigh
90 Matheson Boulevard West
Suite 300
CA-MISSISSAUGA, ON, L5R 3R3
Tel/Tél: +1 905 238 7880
Free: +1 800 665 4831 Institutional
Free: +1 800 868 0441 Home Care
Fax: +1 905 238 7881
E-mail: info.canada@arjohuntleigh.com

AUSTRALIA

ArjoHuntleigh Pty Ltd
78, Forsyth Street
O'Connor
AU-6163 Western Australia
Tel: +61 89337 4111
Free: +1 800 072 040
Fax: + 61 89337 9077

NEW ZEALAND

ArjoHuntleigh Ltd
41 Vestey Drive
Mount Wellington
NZ-AUCKLAND 1060
Tel: +64 (0) 9 573 534
Free Call: 0800 000 151
Fax: +64 (0) 9 573 5384
E-mail: nz.info@ArjoHuntleigh.com

HUNTLEIGH *...performance for life*

Huntleigh Healthcare Ltd. - Diagnostic Products Division
35 Portmanmoor Road, Cardiff, CF24 5HN, United Kingdom
T: +44 (0)29 20485885 F: +44 (0)29 20492520
E: sales@huntleigh-diagnostics.co.uk
W: www.huntleigh-diagnostics.com

Registered No: 942245 England. Registered Office:
ArjoHuntleigh House, Houghton Hall Business Park,
Houghton Regis, Bedfordshire LU5 5XF
© Huntleigh Healthcare Limited 2014

AN ARJOHUNTLEIGH COMPANY, MEMBER OF THE GETINGE GROUP

® and ™ are trademarks of Huntleigh Technology Limited
As our policy is one of continuous improvement, we reserve the right to modify
designs without prior notice.

510901-7