

CLEAN → DISINFECT → RINSE → DRY

General Cleaning Guide

1 Read IFU

Read the products IFU cleaning instructions.



2 Switch Off

Switch off products and disconnect from the AC supply.



3 Clean

Squeeze excess solution from wipes/cloths. Do not allow any fluid to enter the products.



4 Disinfect

Do not use phenolic detergent-based disinfectants, solutions containing cationic surfactants ammonia-based compounds or perfumes and antiseptic solutions.



5 Rinse

After using chemicals ALWAYS rinse off / remove the chemical with absorbent material, *dampened in clean water.*



6 Dry

Dry with a clean cloth.



1 Low Risk Clean

Clean the parts before examining a patient using 'Low Risk' cleaning method below.

2 Deep Clean

After examining a patient clean and/or disinfect the transducers using the appropriate method below:

Low Risk

Use with patients with intact skin and no known infection.

1. Remove soiling, wipe with a mild neutral detergent and then rinse off/remove detergent with an absorbent material dampened in water.
2. Completely dry with a clean cloth.

Medium Risk

The patient has a known infection, skin is not intact and the part is heavily soiled.

1. Follow low risk procedure then wipe with a cloth dampened in Sodium Hypochlorite (1,000ppm).
2. After two minutes rinse off / remove the Sodium Hypochlorite with an absorbent material dampened in water and then dry with a clean cloth.

High Risk

Only use when the part has been contaminated by blood.

1. Follow low risk procedure then wipe with a cloth dampened in Sodium Hypochlorite (10,000ppm).
2. After two minutes rinse off/remove the Sodium Hypochlorite with an absorbent material dampened in water and then dry with a clean cloth.

Please ensure that you check with your facility's local infection control policy and medical equipment cleaning procedures.



Repeated and unnecessary use of concentrated solutions will damage the product. Do not allow Sodium Hypochlorite solutions to come into contact with metal parts.



! Do not use abrasive cloths or cleaners

! Do not use fogging machines

! Do not use automatic washers or autoclaves

Contact:

Huntleigh Healthcare, 35 Portmanmoor Road, Cardiff, CF24 5HN
Phone: +44 (0)29 2048 5885 Fax: +44 (0)29 2049 2520
Web: www.huntleigh-diagnostics.com Email: sales@huntleigh-diagnostics.co.uk

HUNTLEIGH
A MEMBER OF THE ARJO FAMILY